GE VCT 64 SLICE PROTOCOLS

otocol I	Name:	Cl	HEST/	ABDOM	EN/PE	LVIS 5 t	to 10 Y	RS CH	IILD				
NDICATIONS:			Evaluation of child for trauma, chest or abdominal pain, cancer, or abnormal labs.				LABEL:				SCOUT*, CHEST*. The * means anything after these beginnings will hang correct in		
CONT	RAST:		1cc per pound of nonionic contrast. 1.0-2cc/sec. 24ga or better IV.				RECON 1:				.625mm thick/.625 mm interval Std recon at 400ww/30wl. With cor and sag 5mm avg DMPRs.		
RAL CC	NTRA	ST:	2-3 cups of Gastroview is drank the hour prior to scanning.				RECON 2:				5mm thick / 5 mm interval. Storecon 400ww/30wl.		
ATIENT OSITION CAN AR	NING A	AND	Patient supine. Feet first. Landmark above chest. Cuts are taken from above lungs through a few cuts into the iliac crests or pubis if pelvis is also ordered. Same DFOV as previous chest or abdomen CTs.				ADDITIONAL POST PROCESSING:				None		
ENERAL EXAM UIDELINES:			5 x 5 mm axial images. 300 DFOV.				SEND EXAM:				Send scout, 5mm axials, and s and cor images to pacs		
RIGGER ONTRA	OR	AY·	40 Second delay set or remove delay if hand injecting.				ARCHIVE:						
ECHNIÇ UIDELI	UE		100 kvp. Auto ma on.				BILL AS:				Usually chest abdomen pelvis with contrast CT (check dr.'s order)		
				SCOU	JT PRC	TOCO	L SET	IN SC	ANNE	R			
Scan Type		Start Loc		End Loc		kV		mA		Scout Plane		Voice Light Times	S
Scout		S0.00		I500.00		100		10		180		2	
		Above chest		Pubis									
			A	XIAL IN	MAGE :	 PROTO	COL	 SET IN	ISCAN	NER			
Scan Type	Start Loc	End Loc	Thick Speed	Interval (mm)	Gantry Tilt	SFOV	kv	ma	DFOV	Show Recon1	Show Recon2	Show Recon3	Direct Vis
Helical Full 0.5 sec rotation	S0.0 Above chest	I200.0 Below Pubis	5 27.50 1.375:1	5	S0.0	Ped Body	100	275 31.72~ Auto ma on DR 20%	30.0	5mm Std 400ww 30wl Send to Pacs	.625m m Std 400ww 30wl For	0	On 5mm Cor and Sag avg mprs